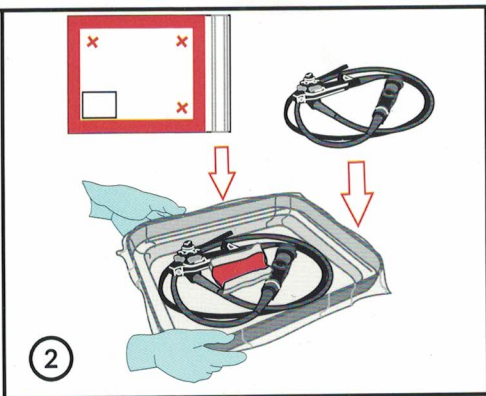
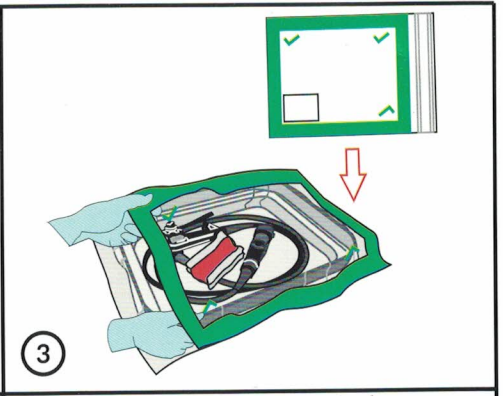
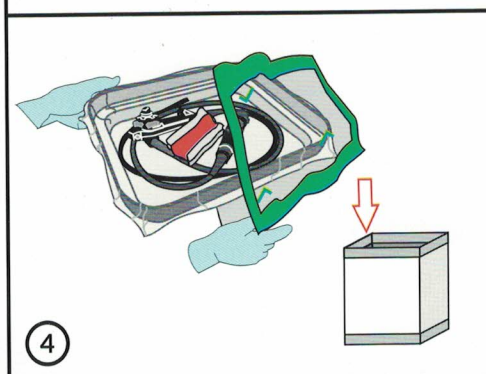
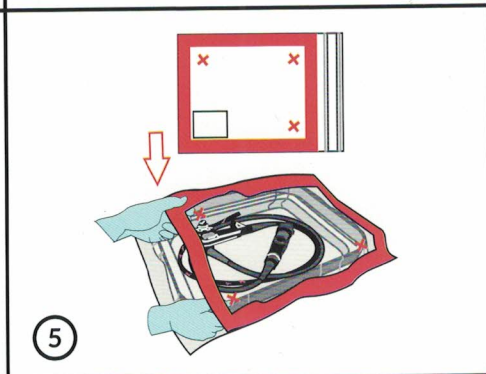
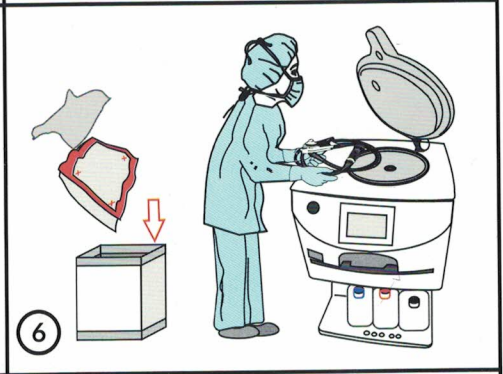
 <p>1</p>	 <p>2</p>	 <p>3</p>
<p>Spread bottom cover in a clean tray.</p>	<p>Place disinfected endoscope and red EzeUse bag on the bottom cover in the tray.</p>	<p>Place the tray in green EzeUse bag and close the opening. Transport to procedure room.</p>
 <p>4</p>	 <p>5</p>	 <p>6</p>
<p>Take endoscope out and discard green bag according to hospital protocol.</p>	<p>Place contaminated endoscope on bottom cover in the tray. Place the tray in red EzeUse bag and close the opening. Transport to disinfection site.</p>	<p>Take endoscope out and discard the bottom cover and red bag according to hospital protocol.</p>